



1. Allow the cup to air dry in a protected space.
2. Inspect for damage and cleanliness/mineral deposits.
3. If damaged, remove cup from use.

STEP 3

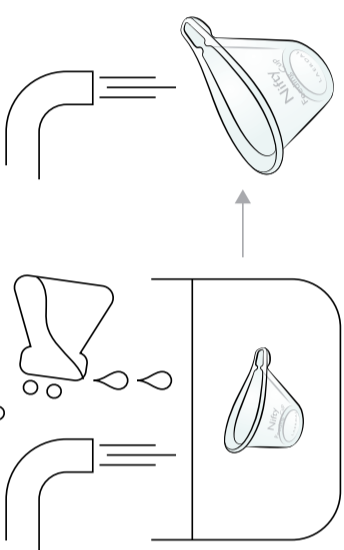


c) Chlorine (0,5%)

1. Use a fresh solution of 0.5% chlorine.
2. Immerse cup completely in the solution for 20 min.
3. Shake off and empty chlorine solution from the cup.
4. Rinse the cup thoroughly in clean water.
5. Repeat 3 times with fresh water to remove any chemical residue.

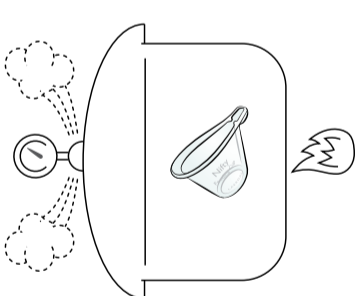

b) Steam Autoclaving

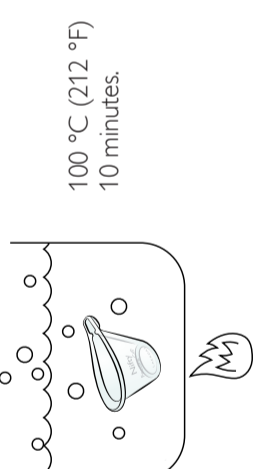


100 °C (212 °F)
10 minutes.

a) Boiling

Perform one of the listed methods:

STEP 2

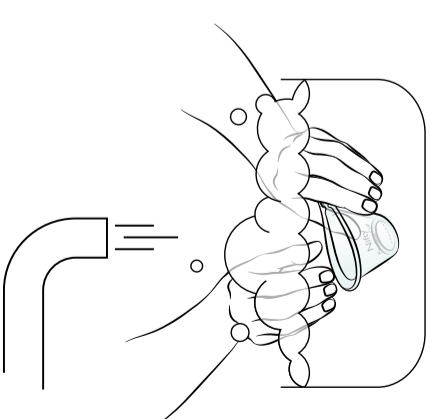

1. Wash in a clean tray with clean water and mild soap.
2. Scrub the cup to remove milk and all other residue.
3. Rinse in clean water to remove soap residue.
4. Repeat above steps until clean.


STEP 1

Reprocessing Instructions

EN

Nifty Feeding Cup

User Guide

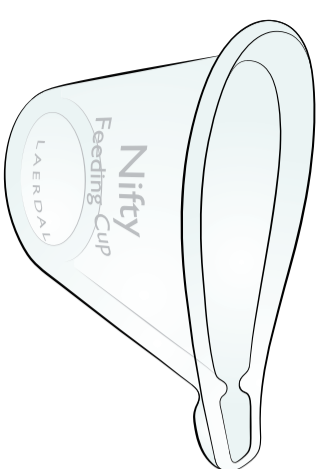

Intended Use

Nifty Feeding Cup is a short-term use product for feeding breastmilk to newborns with breastfeeding difficulties.

⚠ Caution:

The cup should be reused provided proper reprocessing procedures are followed between use with different infants. It must be reprocessed before first use. Refer to Reprocessing Instructions.

Design Collaboration

Nifty Feeding Cup has been developed and designed through a collaboration between the University of Washington, Seattle Children's Hospital, PATH and Laerdal Global Health.

Specifications

Symbol	Definition
	Recyclable
	For food contact according to regulation 1935/2004
	Not made with natural rubber latex

Specifications	
Material	Silicone rubber
Volume	5 to 40 ml
Mass	23.5 g
Max. Dimensions	51 x 82 mm
Operating Temperature	-18 °C to 50 °C (0.4 °F to 122 °F)
Storage Temperature	-40 °C to 60 °C (-40 °F to 140 °F)

Laerdal Global Warranty

Refer to the Laerdal Global Warranty for terms and conditions. For more information visit www.laerdal.com

© 2016 Laerdal Medical AS. All rights reserved.

Laerdal® is a registered trademark of Laerdal Medical AS.

Manufactured in China for:

Laerdal Medical AS
 P.O. Box 377, Tanke Sjølandsgate 30, 4002 Stavanger, Norway
 Tel.: +47 51 51 17 00
 Fax: +47 51 52 35 57
 Email: laerdal.norway@laerdal.com

Printed in Norway

20-10385 Rev A

Nifty Feeding Cup

