


SimMan 3G Trauma

Quick Setup Guide

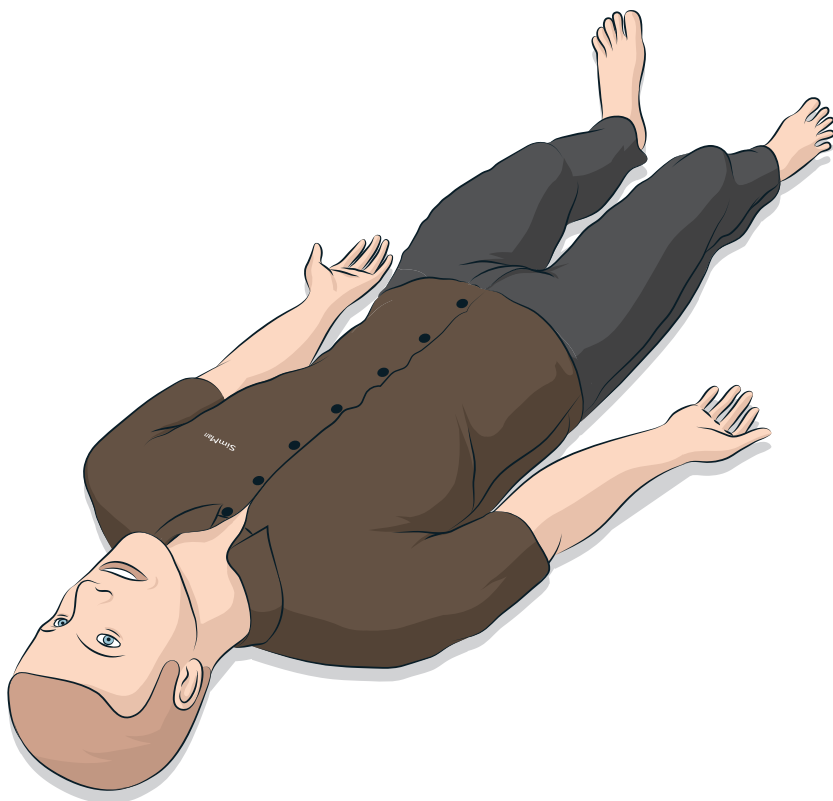


This Quick Setup Guide describes how to assemble and setup your SimMan patient simulator. For information about how to use SimMan, please refer to LLEAP helpfiles and SimMan User Guide.

Visit www.laerdal.com/download to download the latest User Guide and software.




COMPONENTS


Torso

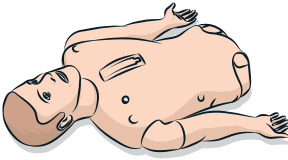


Legs

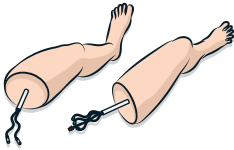

Amputated Arms

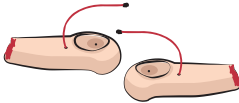

Amputated Right Leg Skin

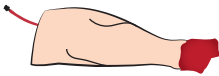


Amputated Left Leg



Clothes



Male & Female Genitalia

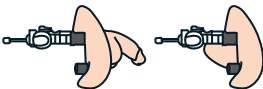

User Guide & Important Product Information



Hard Upper Teeth



Defib ECG Tightner



ECG Adapter Electrode



Lung Compliance O-ring
x 10



Fluid & Blood Refill Bottles



Wounds Kit



Chest Drain Pleura
x 6



Blood Concentrate



Talcum Powder



ID Band



Airway Lubricant



Pinch Clamp




Headset



Blood Pressure Cuff



Inline Filter Blood
x 2

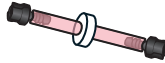

Keyboard & Mouse



IV bag



IM Leg Foam Pads




SpO2 probe



Neck Skin
x6



Pneumothorax

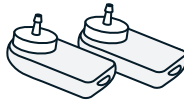

Ethernet Cable



Tibia Pads & Bone
x3



Cricothyroid Tape




Tools



IM Pads



Mounting Hardware

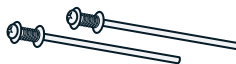

External Power Supply



Sternal io Pads & Bone
x 4



SimMan Arm Adapter




Battery Charger
x 2




1



CHARGE PC BATTERIES




Patient Monitor



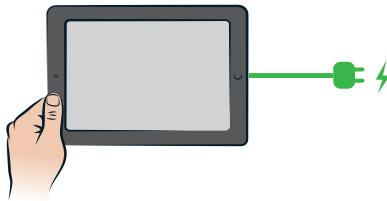


Insturctor PC

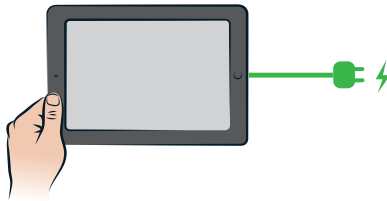

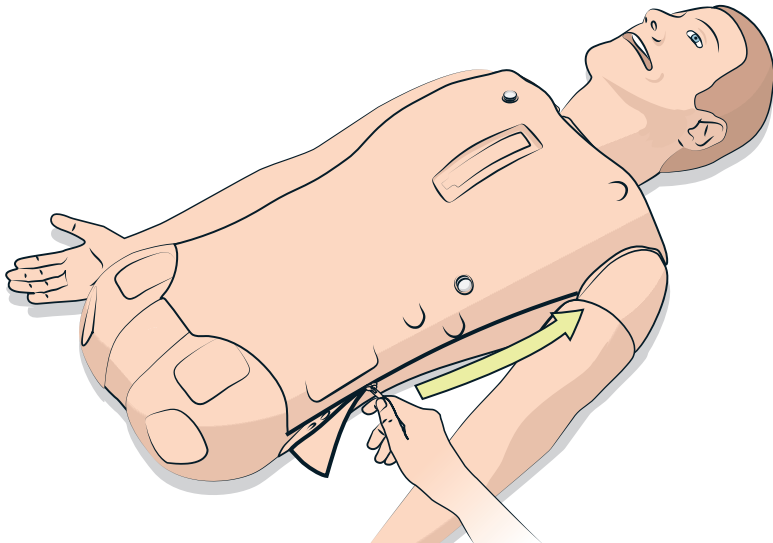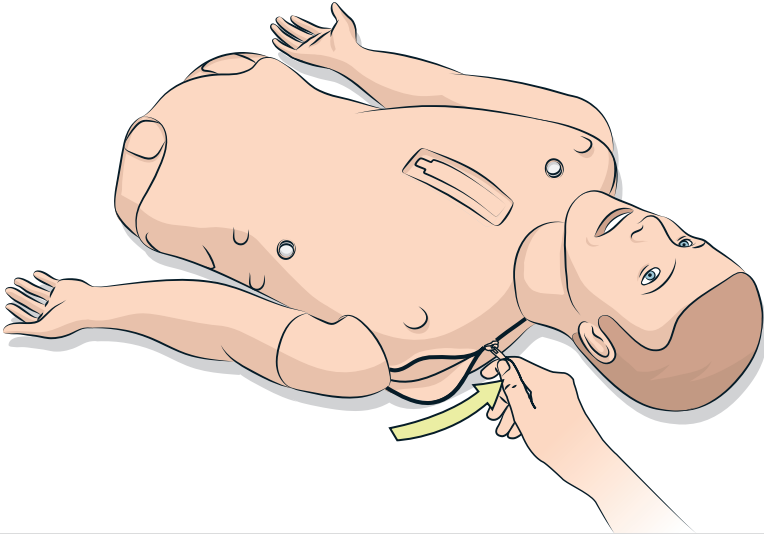


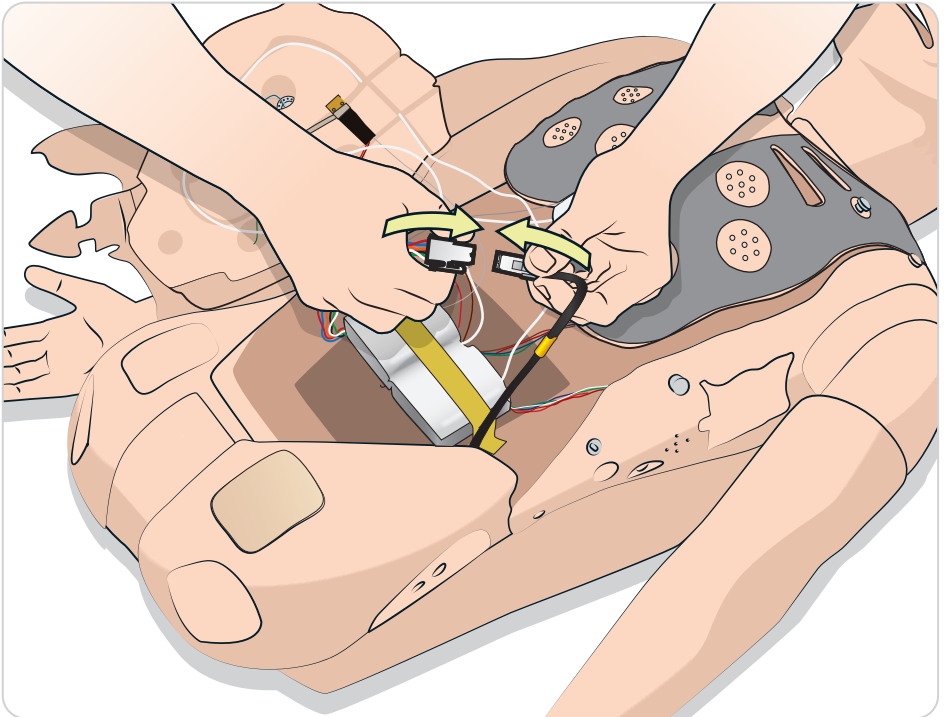
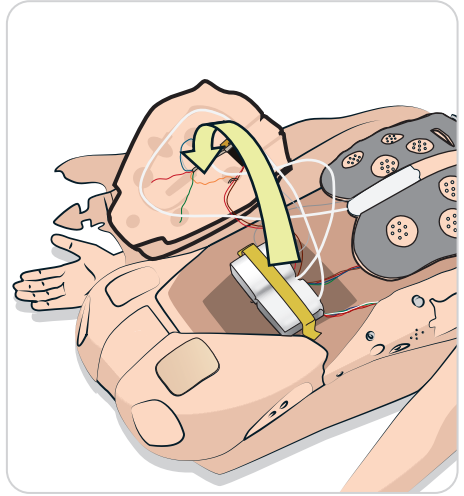
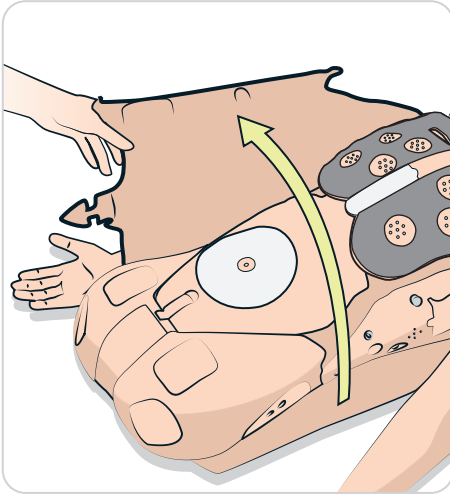


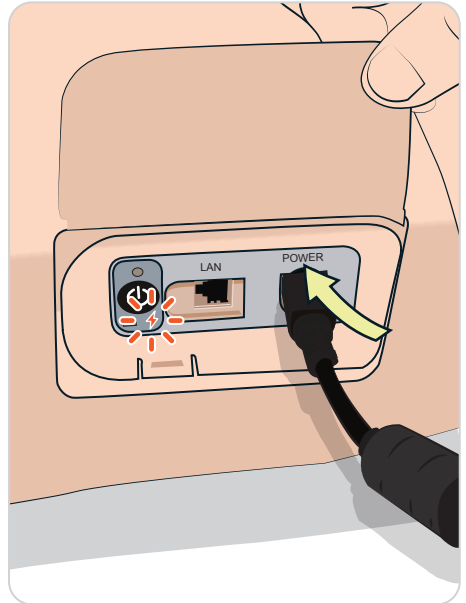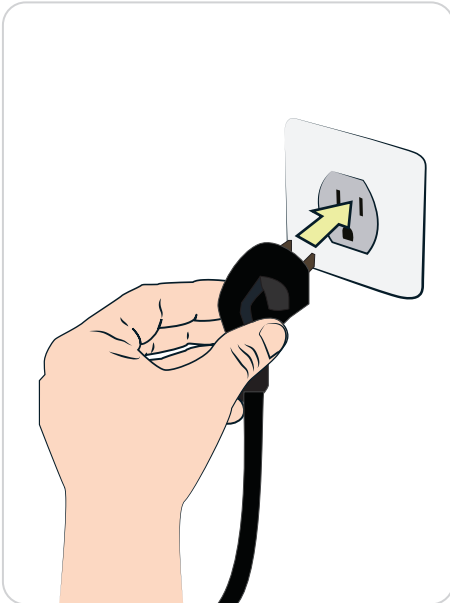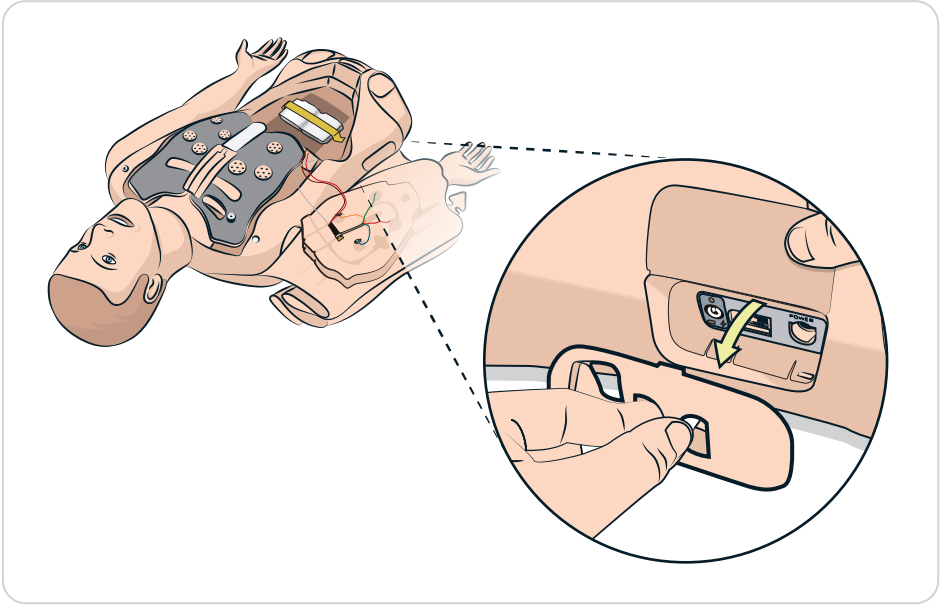

5


ATTACH LEFT LEG



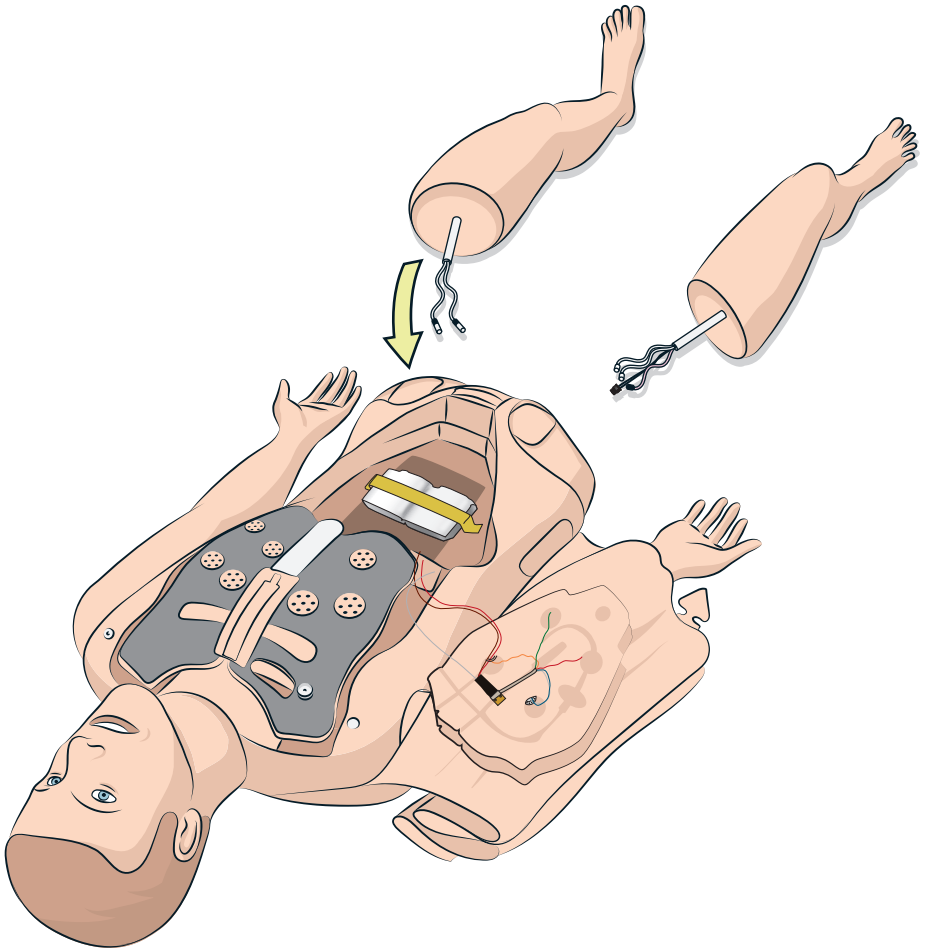


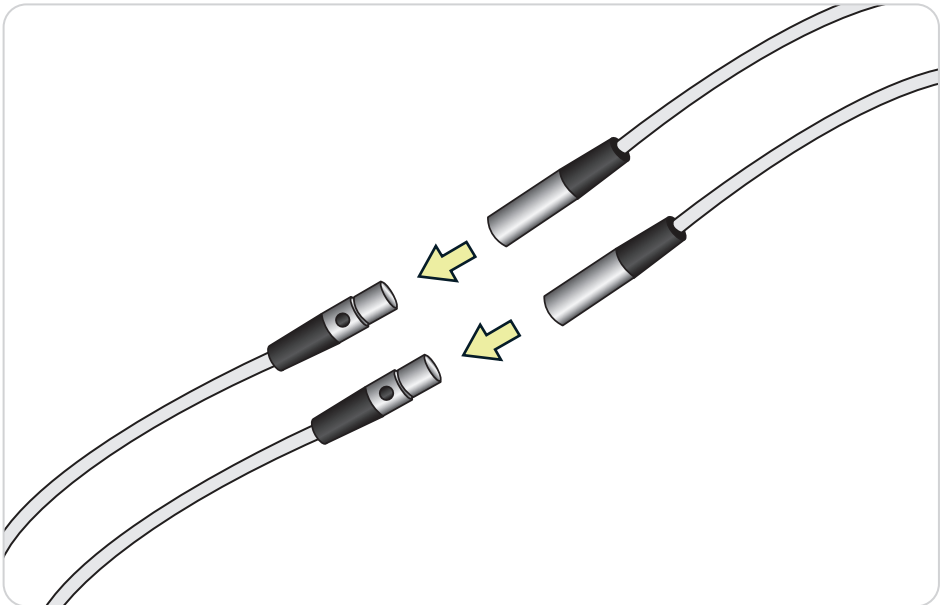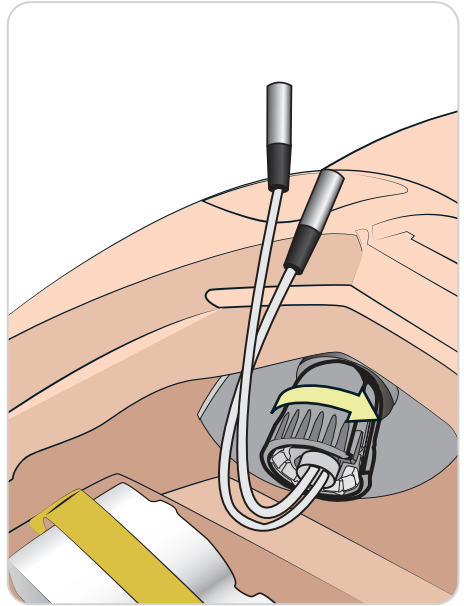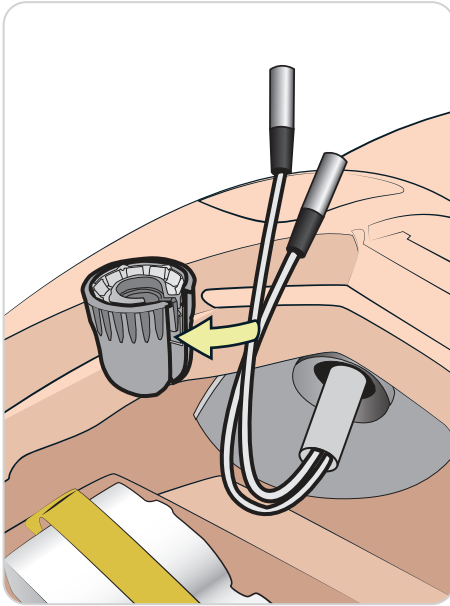

6

ATTACH RIGHT LEG

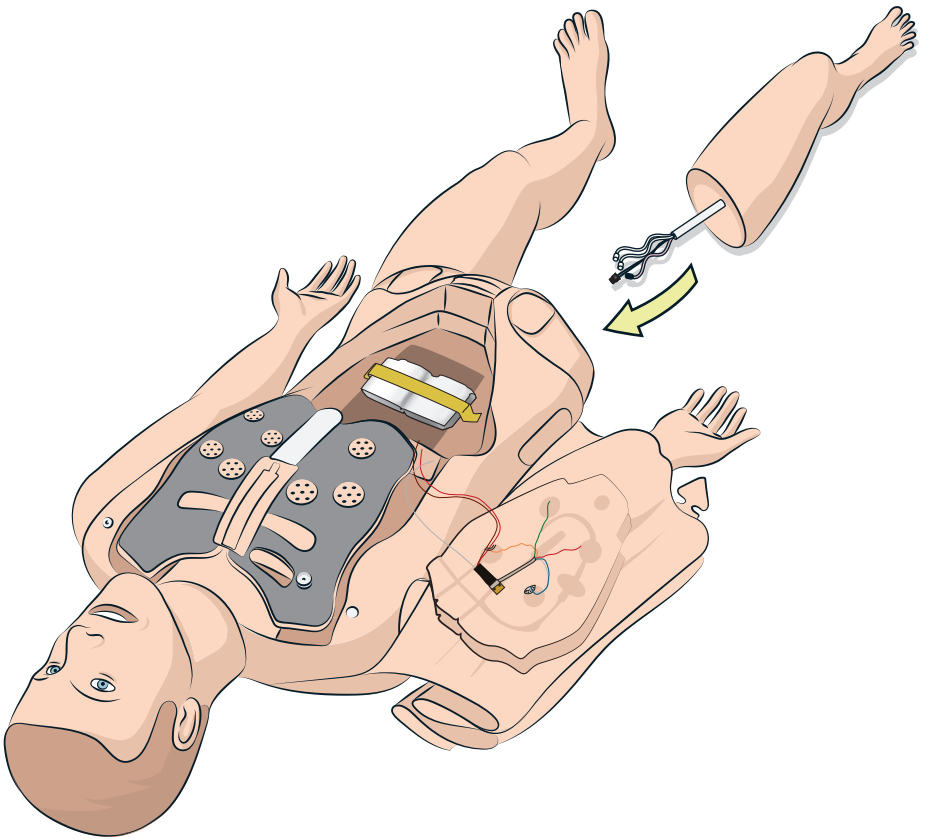

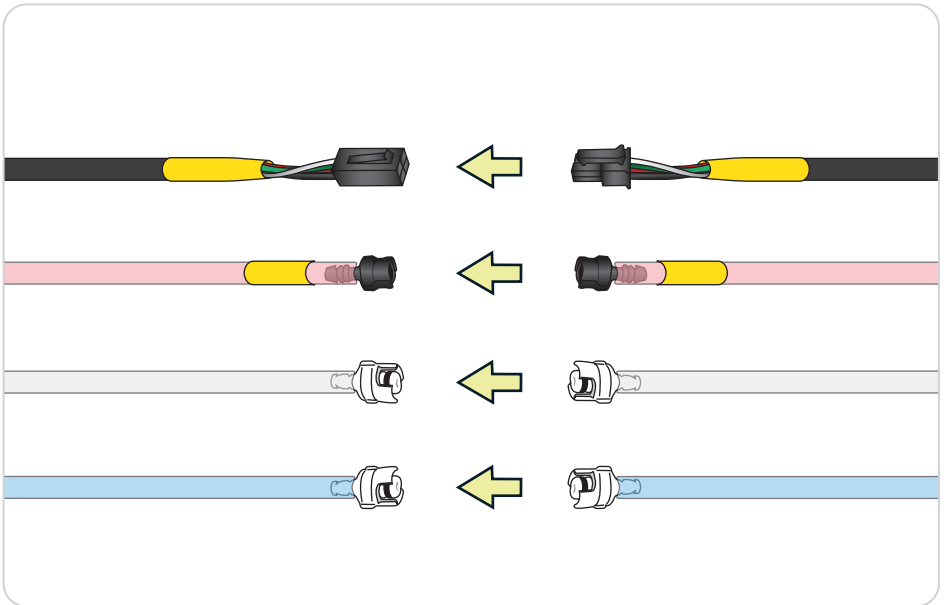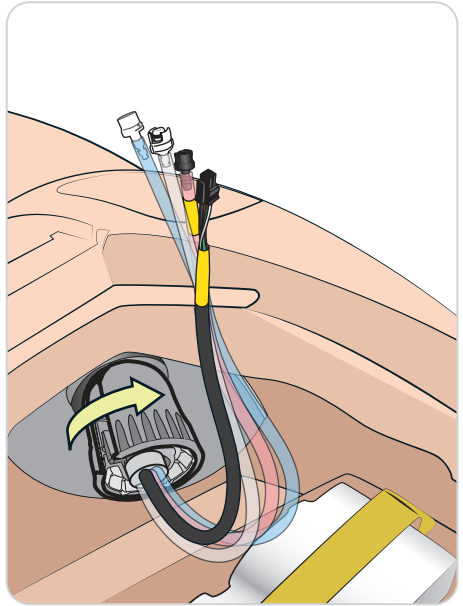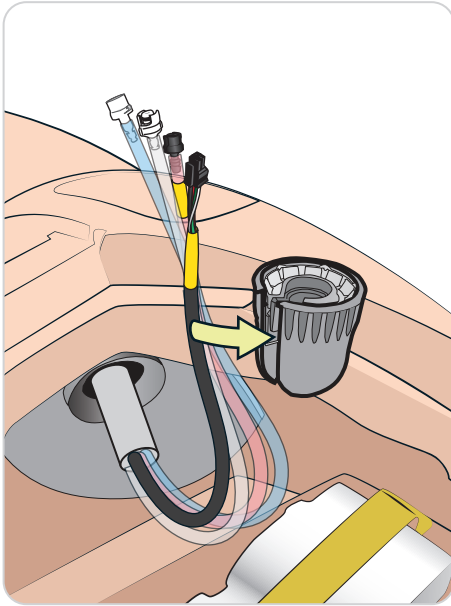


7_A

WIRELESS SETUP

Headset




Instructor PC

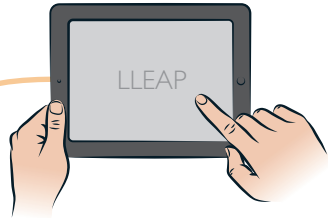


SpO2



Patient Monitor

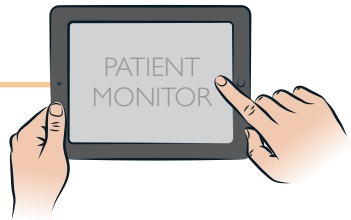

USB cable —

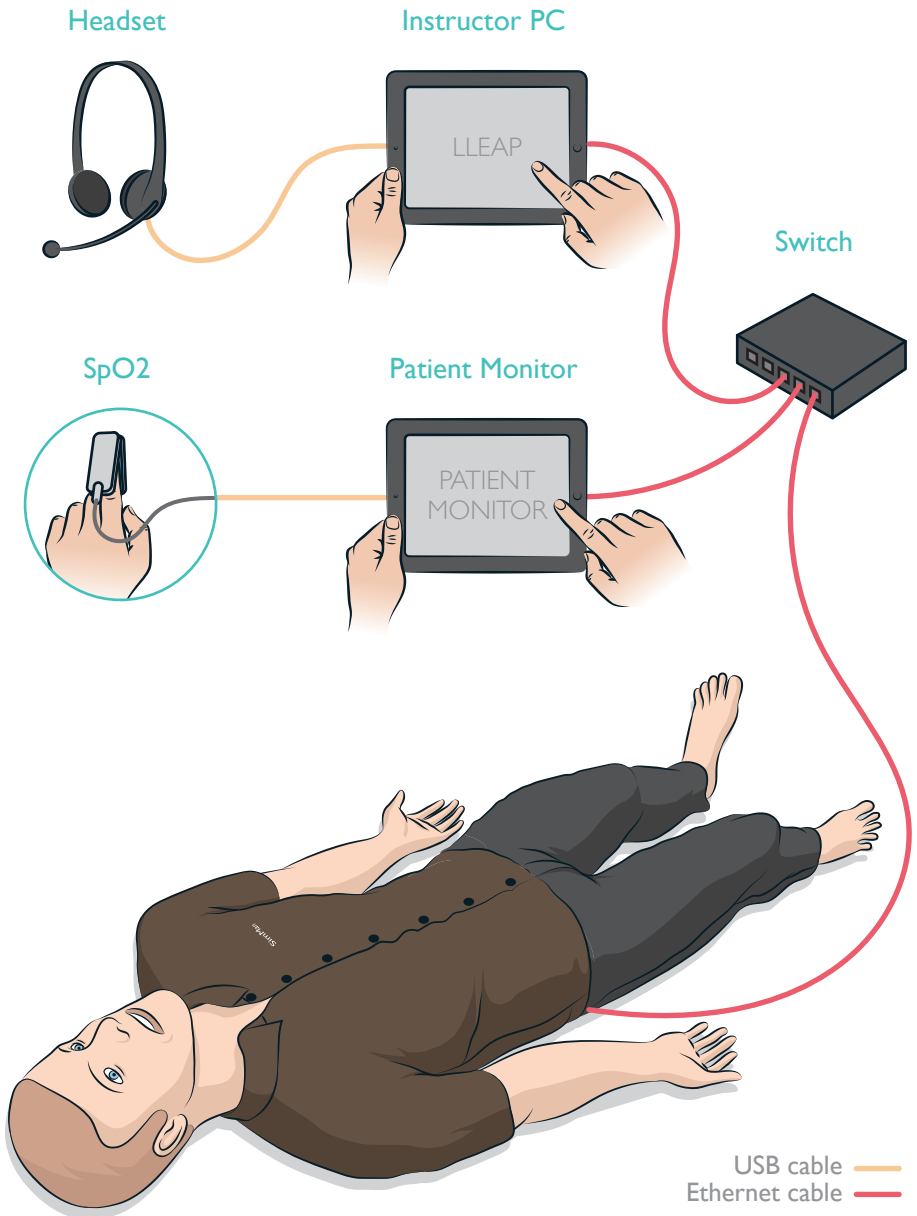

© 2016 Laerdal Medical AS. All rights reserved.
Printed in Norway

20-08044 Rev B

www.laerdal.com



Laerdal
helping save lives