
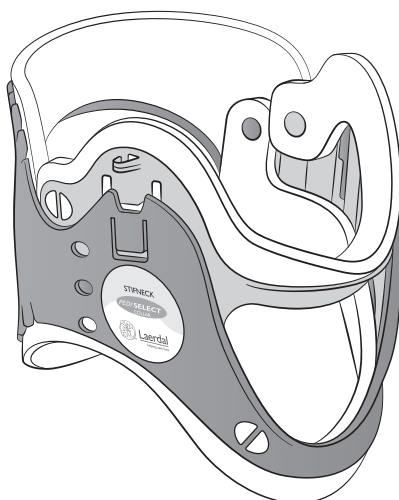
# Stifneck Select

Select and Pedi-Select Extrication Collar

## User Guide








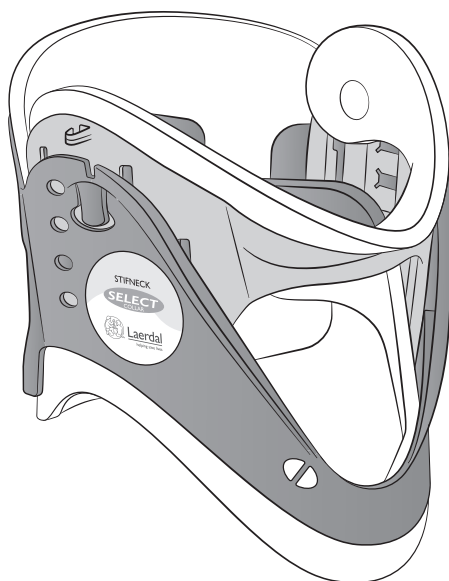
- 98001005 Stifneck Select (J) (Qty:1)
- 98001205 Stifneck Select, green (J) (Qty:1)
- 98002005 Stifneck Pedi-Select (J) (Qty:1)
- 98991040 Stifneck Select, Private Label (Qty: 1)
- 98992040 Stifneck Pedi-Select, Private Label (Qty: 1)

Rx ONLY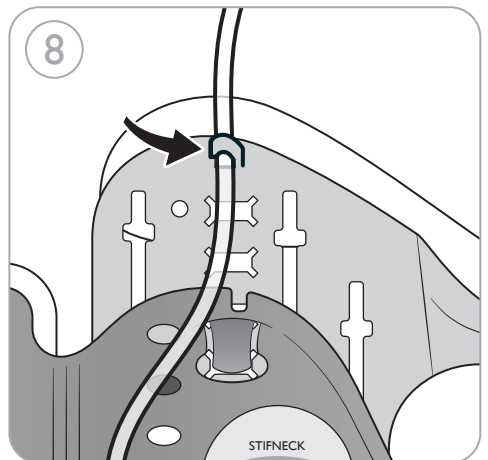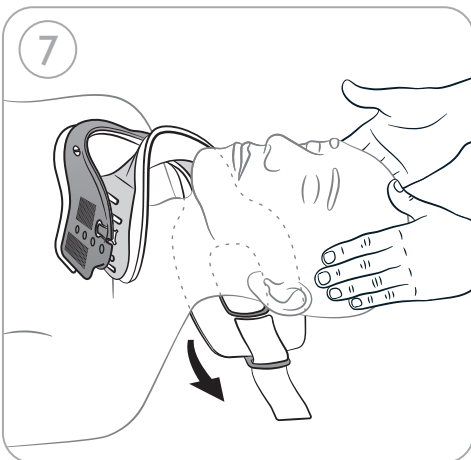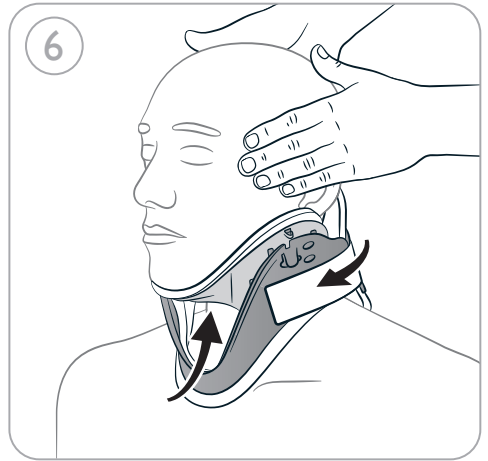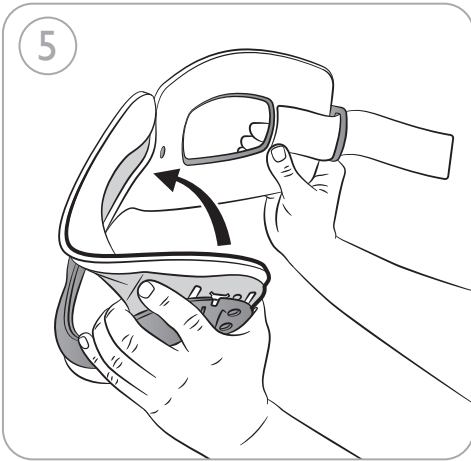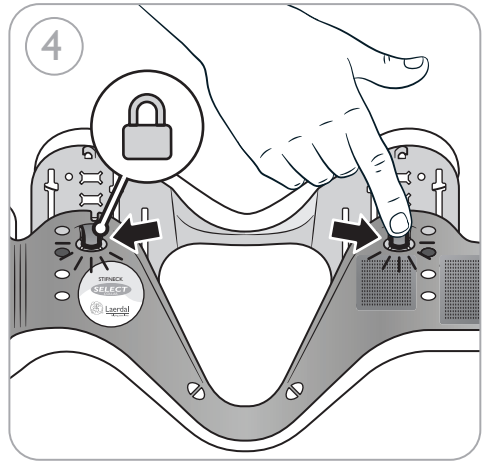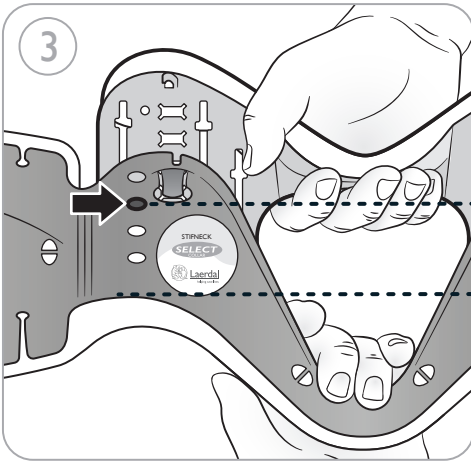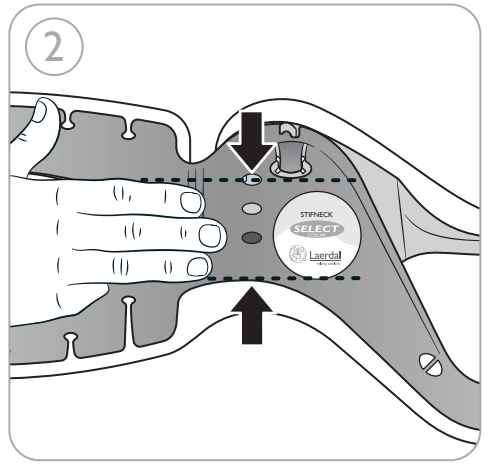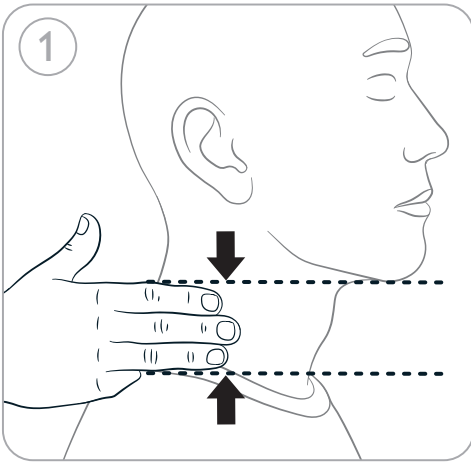

## Intended Use

This product is intended to support cervical spine in a neutral position during transportation, in combination with other cervical and full body immobilization devices (CSIDs and FBSIDs).

### Notes

- In cases of a suspected spinal cord injury, proper neck immobilization is only one part of total immobilization.
- It is imperative that the patient is properly immobilized to prevent any movement of the spinal column.
- Should any serious malfunction, undesirable incident with, or deterioration in the functionality or performance of the device occur, contact Laerdal promptly. The competent authority where the incident took place and/or the device was used should also be notified.

### Warnings

- Stifneck Select Collars should only be used by healthcare providers.
- Improper use may result in injury or permanent disability. Always follow local spinal immobilization protocol.
- Use correct collar size. An oversized collar may hyperextend a patient's cervical spine. An undersized collar may not provide appropriate stability.
- Do not adjust the Select or Pedi-Select collar on patient.

## User Instructions

- Measure patient in neutral head position from shoulder to chin. Align head to neutral or "eyes forward" position unless contraindicated by local protocol.
- Match collar size to patient as shown in step 2 (from plastic edge to size window). Select has 4 positions, Pedi-Select has 3 positions.
- Adjust collar and chin support to size selected in step 2.
- Lock both sides by pressing the two lock tabs.
- Preform the collar.
- Apply collar while manually maintaining neutral head position. Position chin support under chin. If a different size is needed, remove, resize, and re-apply. Pull the collar around the neck ensuring a close fit, while holding the front in place - then fasten.
- For a supine patient, slide the rear panel behind the neck before positioning the chin support.
- Cannula hooks are provided for attaching supplemental oxygen lines.

Technical Specifications	
Storage	Do not store collar in folded position. Store in flat position.
Storage Temperature	-34 °C to 52 °C (-30 °F to 125 °F)
Operating Temperature	-25 °C to 43 °C (-13 °F to 110 °F)
Symbol Glossary	
	This medical device complies with the general safety and performance requirements of Regulation (EU) 2017/745 for medical devices.
	Not made with natural rubber latex.
	Do not re-use.
	Medical Device
	Rx Federal law (USA) restricts this device to sale by or on the order of a physician.

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**Manufactured in China for:**

Laerdal Medical AS  
P.O. Box 377  
Tanke Svilandsgate 30, 4002 Stavanger, Norway  
Tel.: (+47) 51 51 17 00

**Distributed by:**

Canada and Latin America:  
LAERDAL MEDICAL CANADA LTD.  
151 Nashdene Rd., Unit #45  
Toronto, ON, Canada, M1V 4C3  
Tel.: +1 (416) 298-9600,  
Toll free 888/LAERDAL (523-7325)  
ou en français (800) 567-9987  
Fax: +1 (416) 298-8016

**USA:**

LAERDAL MEDICAL CORPORATION  
167 Myers Corners Road, P.O. Box 1840  
Wappingers Falls, New York 12590-8840  
Tel.: (800) 431-1055, +1 (845) 297-7770  
Fax: (800) 227-1143, +1 (845) 298-4545

**Australia**

Laerdal Pty. Ltd.  
8 Stamford Road  
Oakleigh VIC 3166.

**Laerdal do Brasil**

Al.Tocantins 125 – sl 01 – Alphaville  
Barueri / SP / Brazil  
06543-260  
Tel.: +55 11 4193-8007  
e-mail: comercial@laerdal.com

© 2020 Laerdal Medical AS. All rights reserved.

Printed in China